

PRP & PRF

Face & Hair

Rejuvenation

Advanced Module

PURE
R I C H
P A R T

March 2025

Who are we:

PURE
RICH
PART

- * In UK and Ireland since 2016
- * Over 20 years of combines experience on PRP
- * The only UK manufacturer registered with MHRA with Article 12 - UK / EU regulations
- * Comprehensive and cost effective PRP Kit
- * Well recognised brand with presence in UK and Ireland
- * Lifetime support
- * Marketing support
- * 5-star Google Reviews

Why PRP ?



Why PRP ?



BEFORE



AFTER

Why PRP ?



Why PRP ?



Why PRP ?



Why PRP ?



BEFORE



AFTER

Why PRP ?



A



ETE Aesthetic

B


Why PRP ?



Why PRP ?



Healing cascade

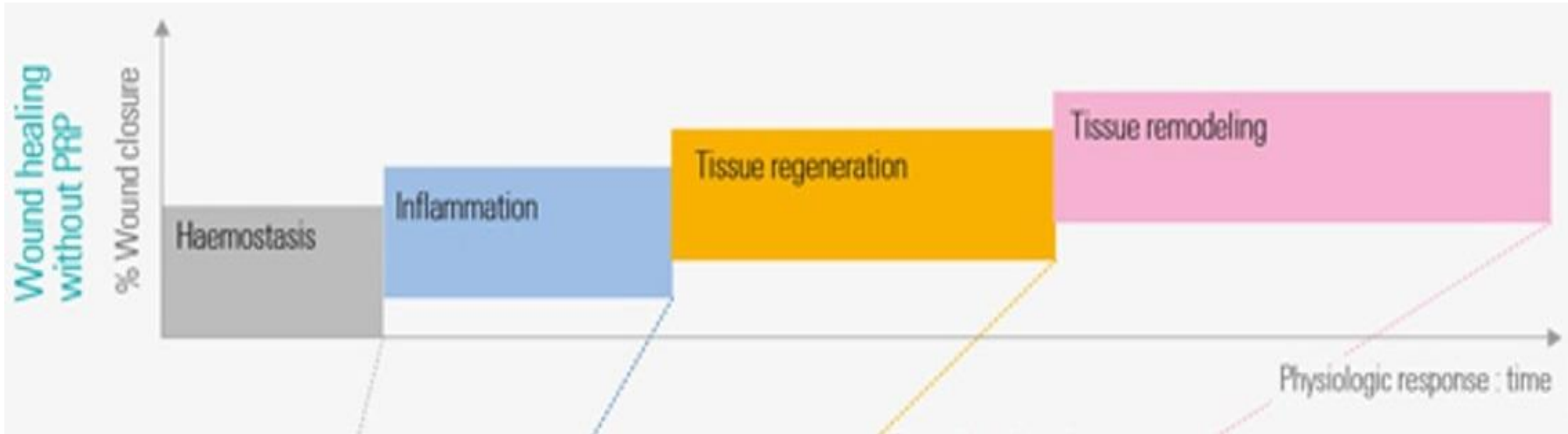

PRP –Platelets Rich Plasma

Wide Range of Applications

CLINICAL:

- Sports medicine
- Traumatology
- Plasma treatments – Covid 19
- Ophthalmic
- Wound healing – diabetic foot
- Dentistry- sinus lift
- Tooth implants

AESTHETIC:

- Skin rejuvenation
- Hair restoration
- Post laser
- Acne & Rosacea
- Acne scars
- Tissue Volumisation – alternative of HA fillers
- Aesthetic gynecology /urology

Current Injection Options

Correction

Dermal fillers/Botox

- FILLERS
 - Mechanical volume correction
- Botulinum toxins leads to Muscular relaxation
- Short term effect

Therapy

Collagen Induction therapy - microneedling

- Biological Cellular action
- Therapy and prevention
- Progressive global action to all tissue
- Long term- enhancement of appearance

PRP mechanism of action

- Tissue trauma
- Hemostasis - Platelets + Leucocytes form 3D mesh → release of GF
- Chemo attraction and migration of macrophages and stem cells into area of trauma
- Stem cells proliferate by mitosis
- Stem cells undergo differentiation process into the specific tissues that were originally injured.

Role of GF

PLATELET



PDGF

**Platelet Derived
Growth Factor**

Cell Growth, new generation and repair of blood vessels, collagen production



VEGF

**Vascular Endothelial
Growth Factor**

Growth and new generation of vascular endothelial cells



FGF

**Fibroblast
Growth Factor**

Tissue repair, cell growth, collagen production, hyaluronic acid production



EGF

**Epithelial
Growth Factor**

Promotion of epithelial cell growth, angiogenesis, promotion of wound healing




TGF-β



**Transforming
Growth Factor**

Growth and neogenesis of epithelial cells and vascular endothelial cells, promotion of wound healing

Role of GF

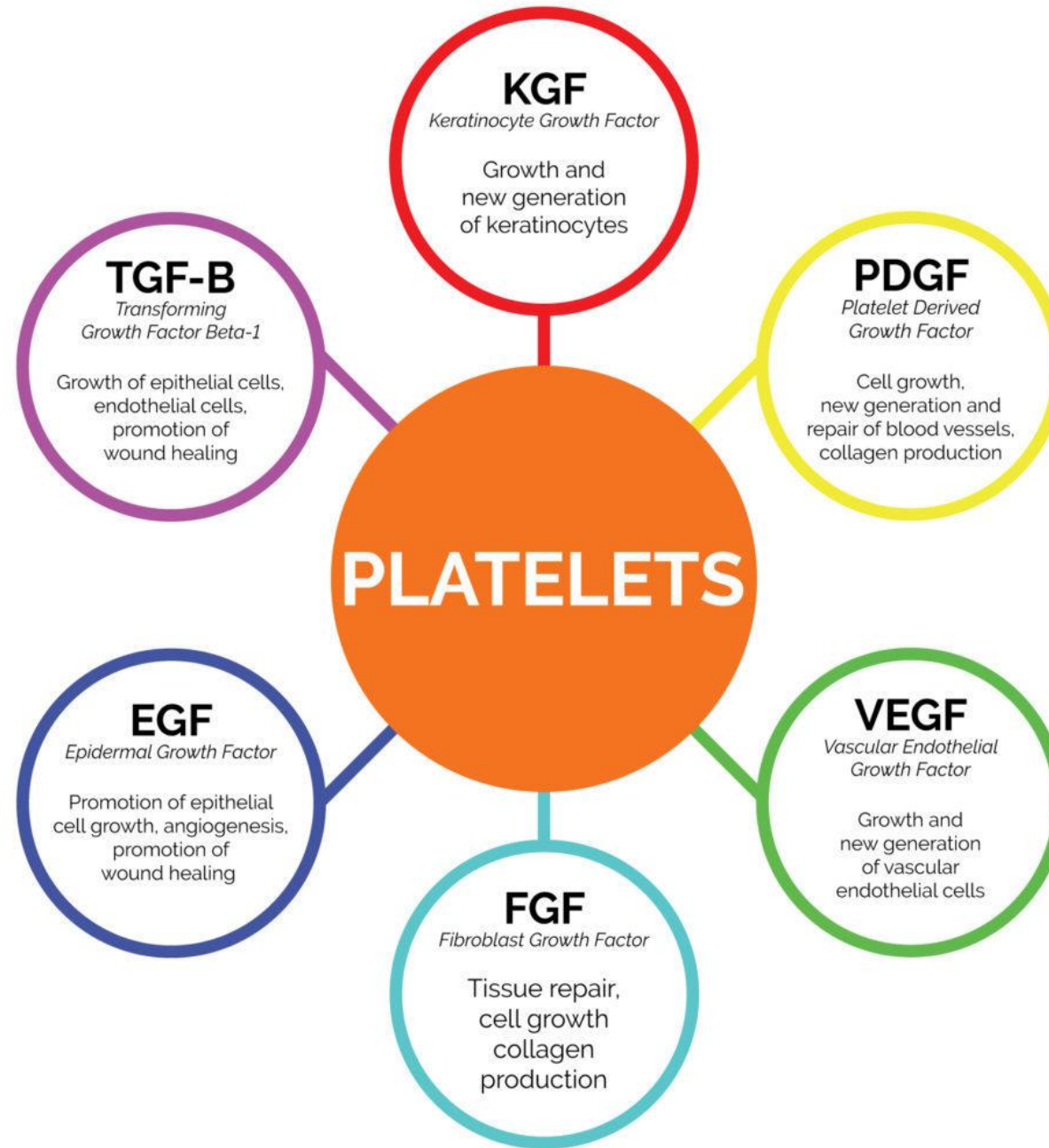


Skin Anatomy:

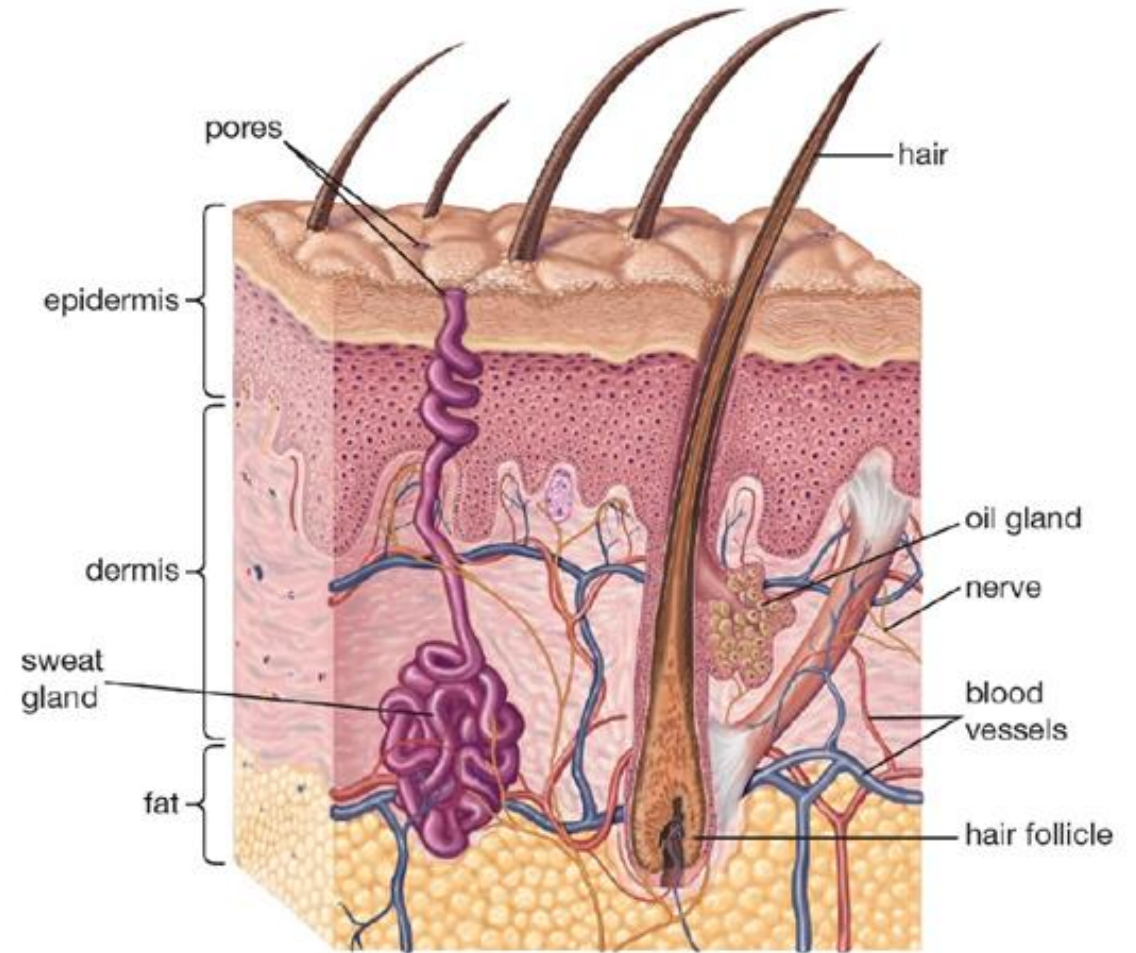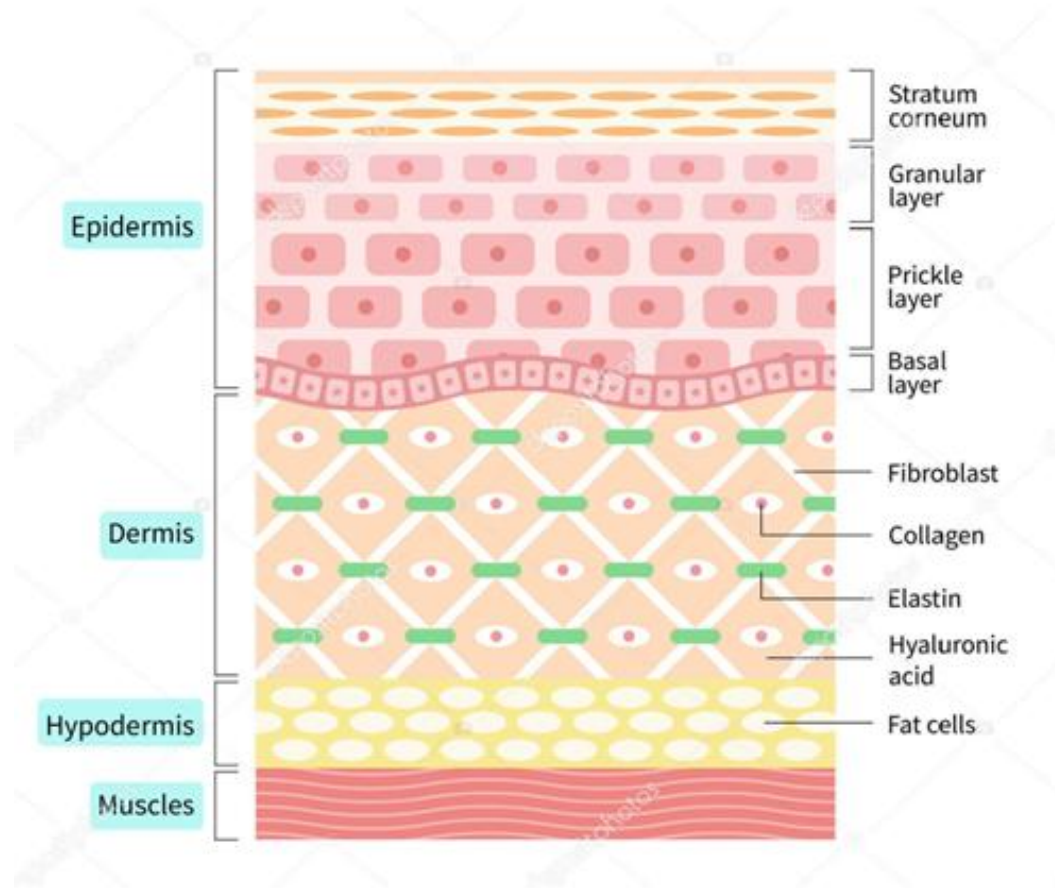


© 2013 Encyclopædia Britannica, Inc.

PRP accelerates of healing cascade

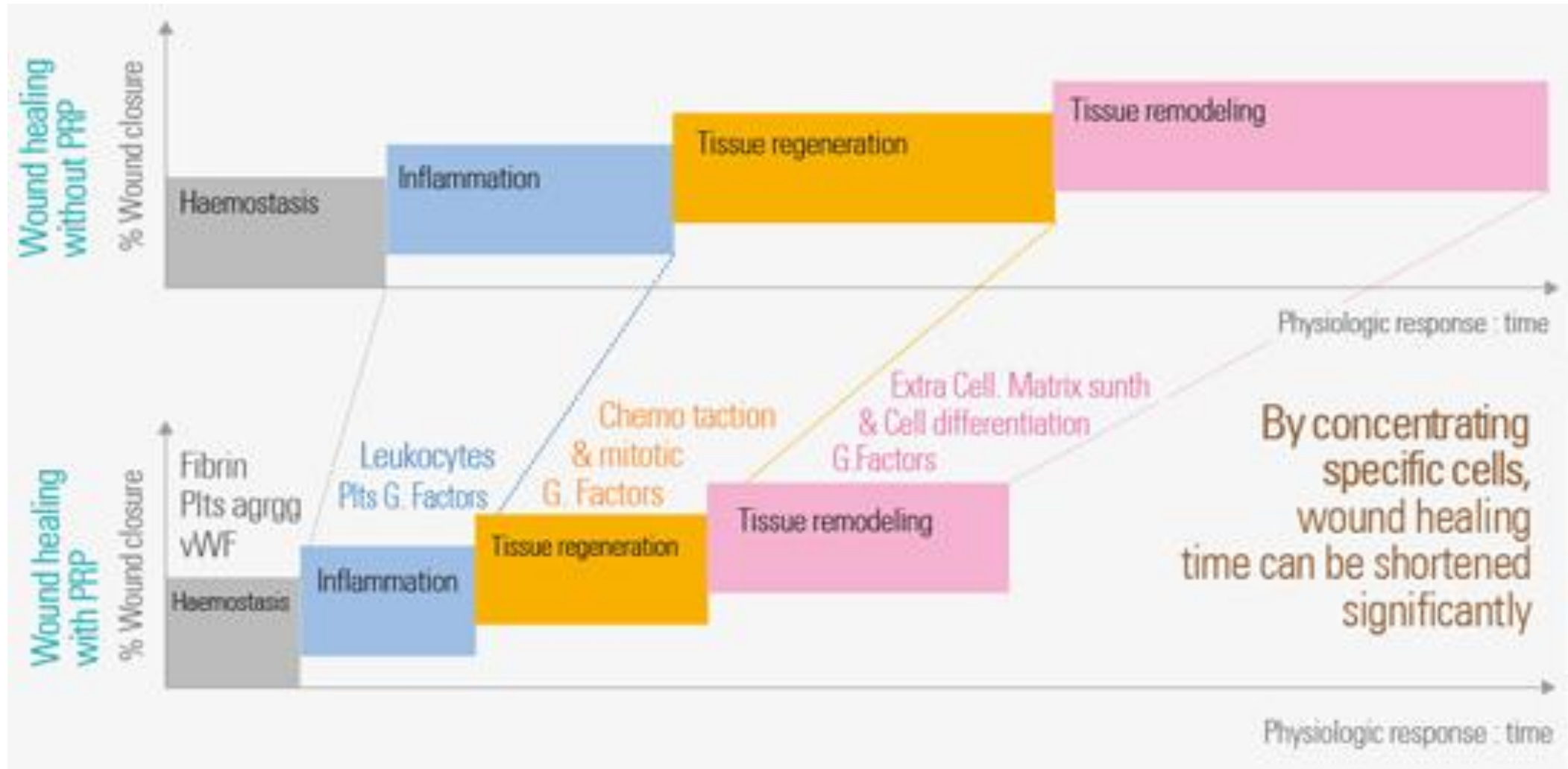

Process of PRP Therapy

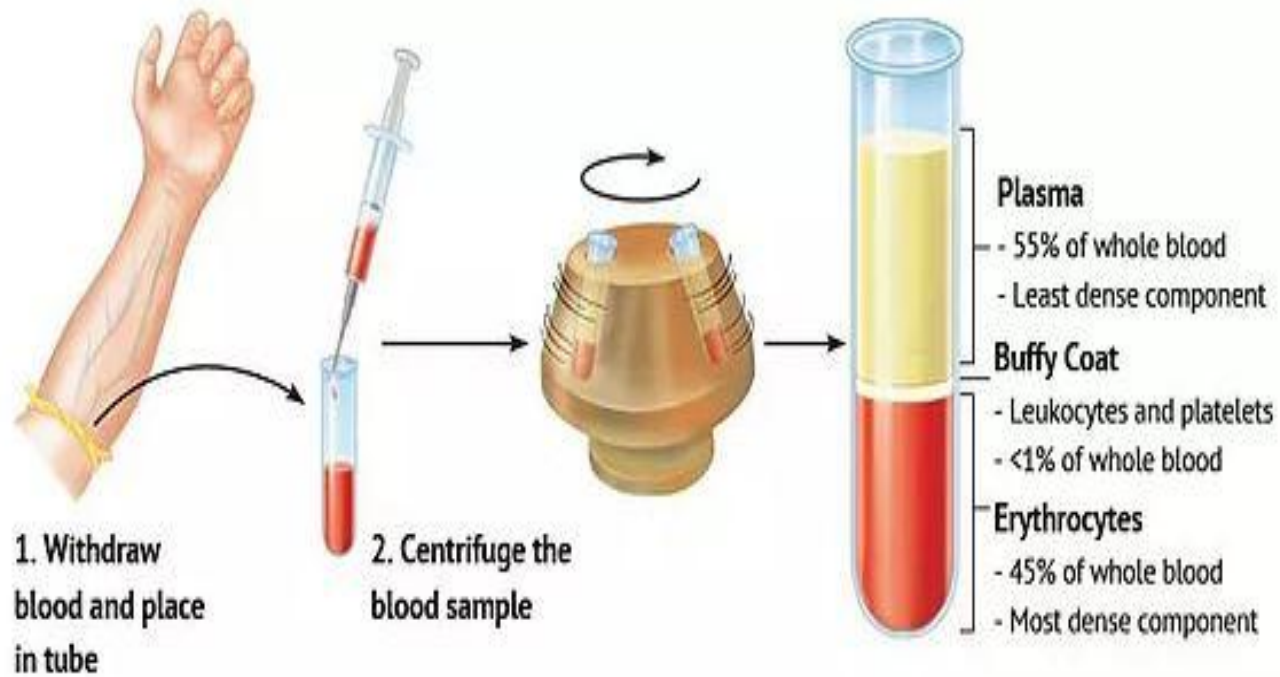

PRP –Where works?

- Skin rejuvenation- facial resurfacing
 - ✓ application- injection alone
 - ✓ Micro needling
 - ✓ Combination with HA, peeling
 - ✓ Hair restoration
 - ✓ Multiple protocols with positive results
 - ✓ Improvement of AGA hair loss

Benefits of PRP

- **Autologous** – own blood, GF-natural substances
 - Free from disease transmission and no hypersensitivity
- Enhances wound healing – regulate mitosis, proliferation & differentiation
- Augment tissue – Fibrin – collagen& elastin, HA
- Angiogenesis – leads to improved tissue oxygenation and nutrition flow
- Adipocyte and Keratinocyte Generation(fat and skin rejuvenation)

PRP and PRF kits

Strict European manufacturing procedures

CE mark – medical device class II

Patented European producer

One spin preparation

Easy to perform

Safe : ONE step close system

Pure AUTOLOGOUS - No additives, activators or GEL

Individually sterile tubes

COST EFFECTIVE - 6 X 10ml tubes with accessories

Full range of tubes (PRP & PRF)

£100



PRP PURE Tubes



- Single spin
- Anticoagulant – Sodium citrate
- 95% PLT harvesting
- No additives for PTL activation
- Activation - Tissue coagulation proteins
- Stem cells bio stimulation



PRP PURE Centrifuge




- **European model**
- **FDA Approved, CE marked, UKCA**
- **Desktop model, designed for small batch application, 4.5kg**
- **Capacity – 6 nests**
- **Speed range 300 – 4500 RPM**
- **Timer from 30sec to 99min run**
- **Brushless DC motor**
- **Gentle acceleration and gentle braking**
- **3 years WARRANTY**
- **£550**

PRF – Platelets Rich Fibrin Matrix

- Lower platelets concentration, lower in volume, includes proteins and plasma
- Fibrin is included and develops a matrix which binds Platelets and GF
- Acts as natural clot with lower level of GF” time released ” over a longer period
- Give more structure to the tissue
- PRF KIT – 4 X 10ml tubes - £40 per set + accessories

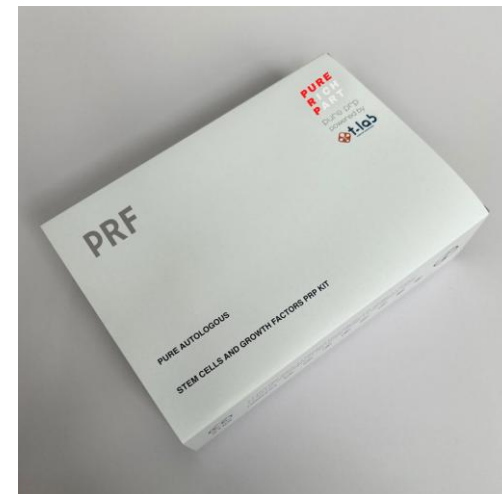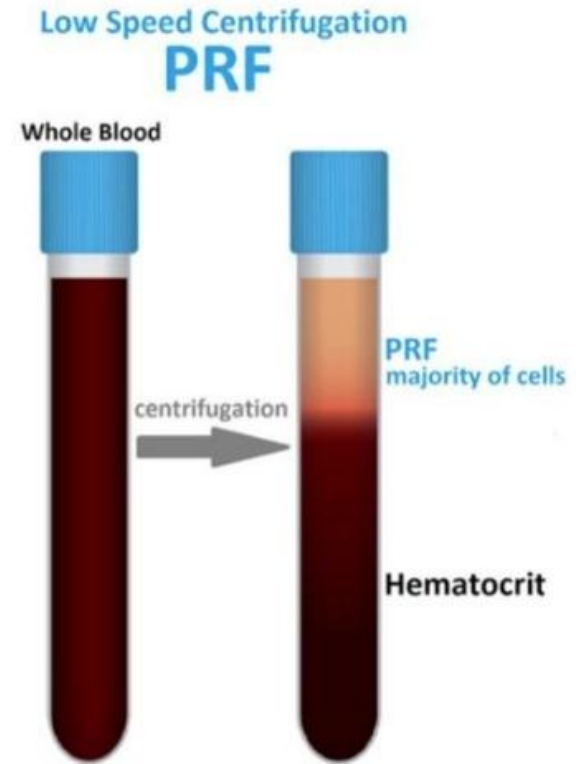

PRF

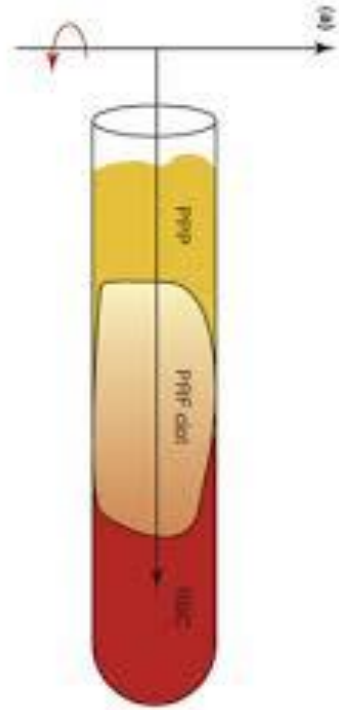

- PLT + WBC **fibrin clot**
- complex fibrin matrix
- Fibrin clot containing the majority of the platelets and white blood cells
- Accelerate the healing of soft and hard tissue
- Tissue-engineering scaffold

Indications:

Dentistry:

oral surgery , implants, bone grafting, periodontal, surgery, extractions


Aesthetics:

Topical + Microneedling

PRF – Platelets Rich Fibrin / IPRF



Pros



- Autologous
- Slower GF release
- Forms a scaffold
- Benefits Dental work
- Benefits Microneedling

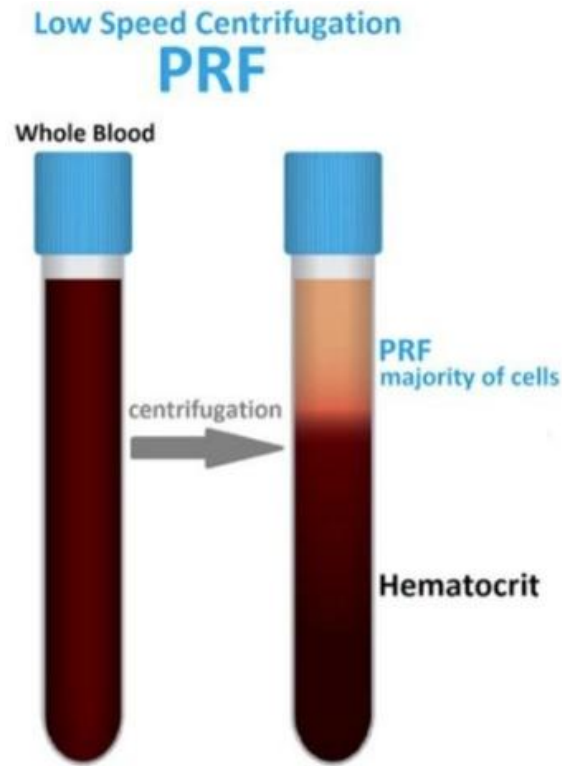



Cons

- Low in volume
- Clots quickly, fast work needed
- More expensive than PRP
- Massive Inflammatory reaction
- Doesn't last long
- Potential Risk of thrombosis if used incorrectly

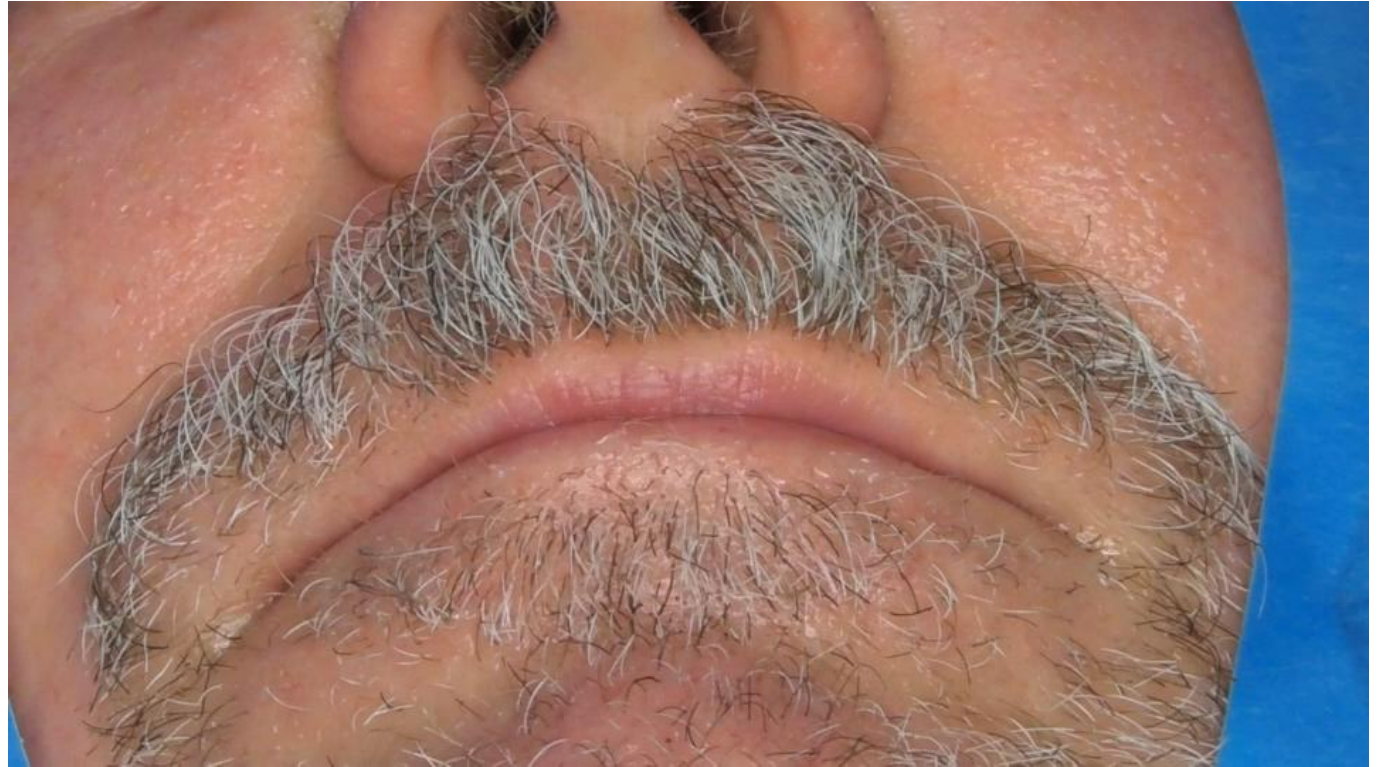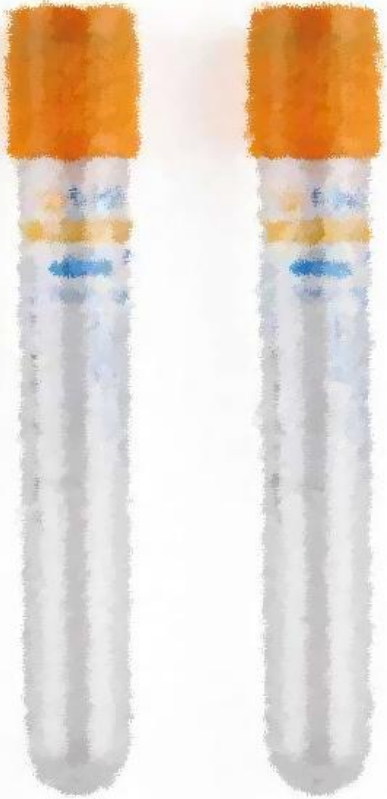

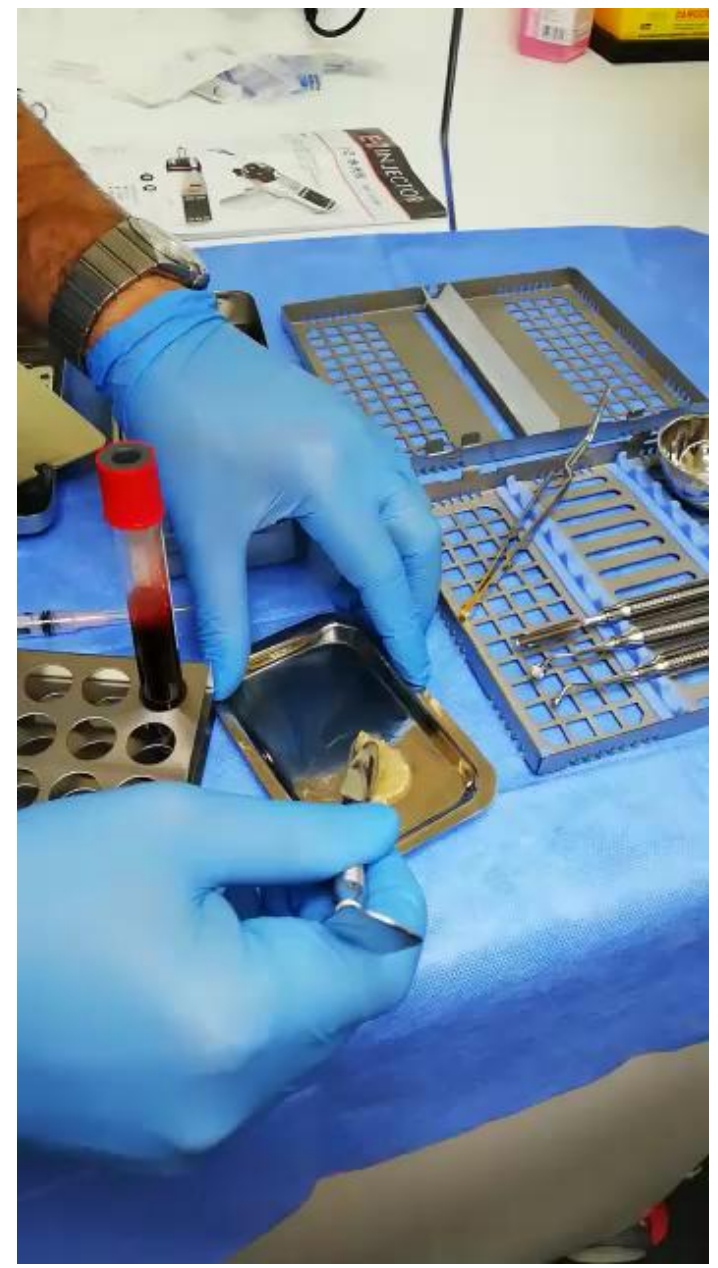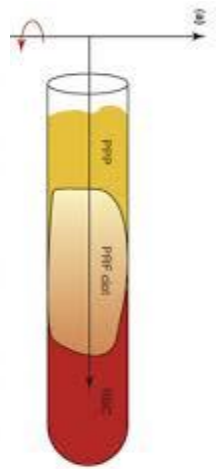


Skin Anatomy:

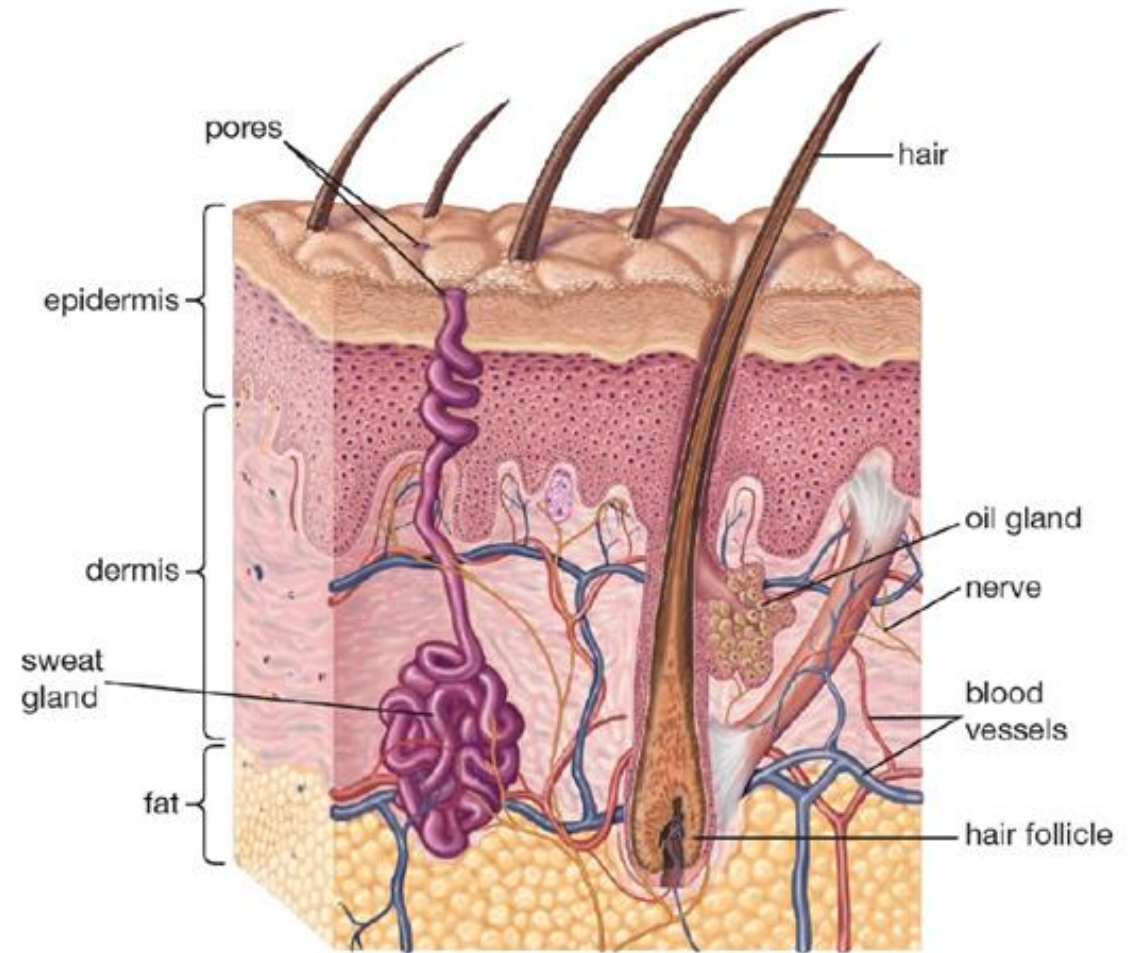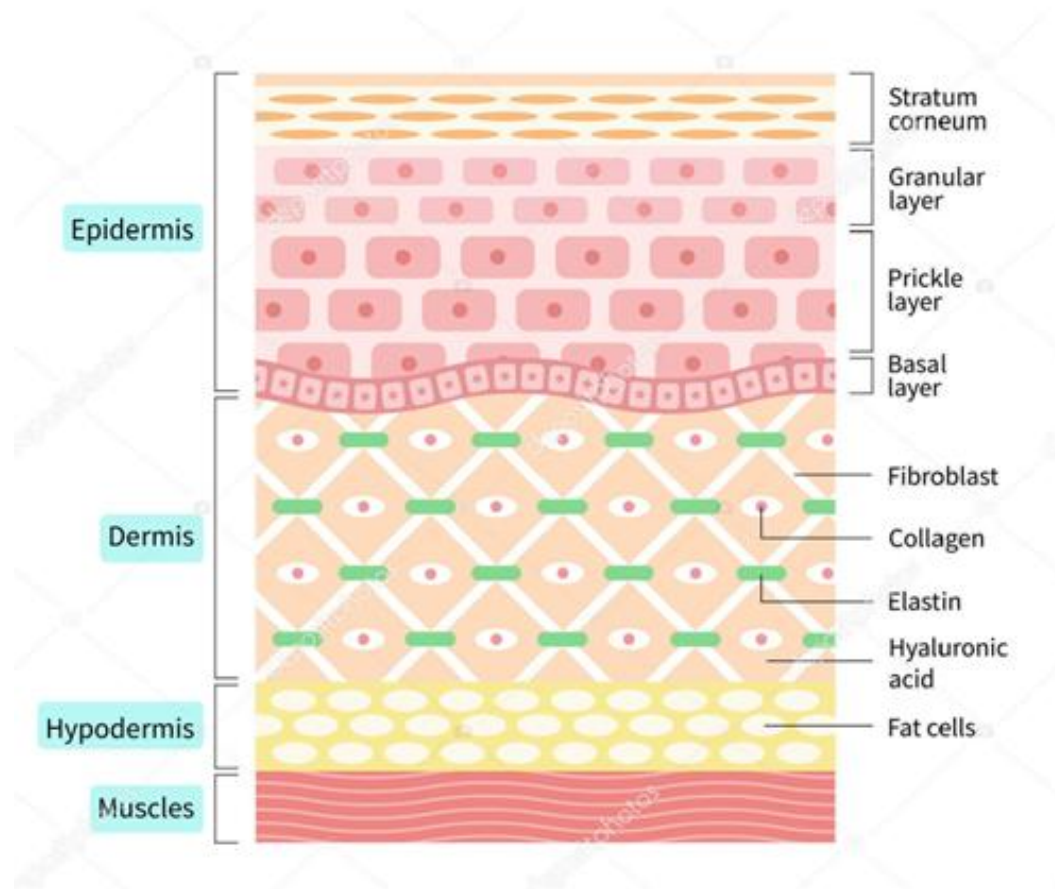

© 2013 Encyclopædia Britannica, Inc.


Age related skin structure deformation

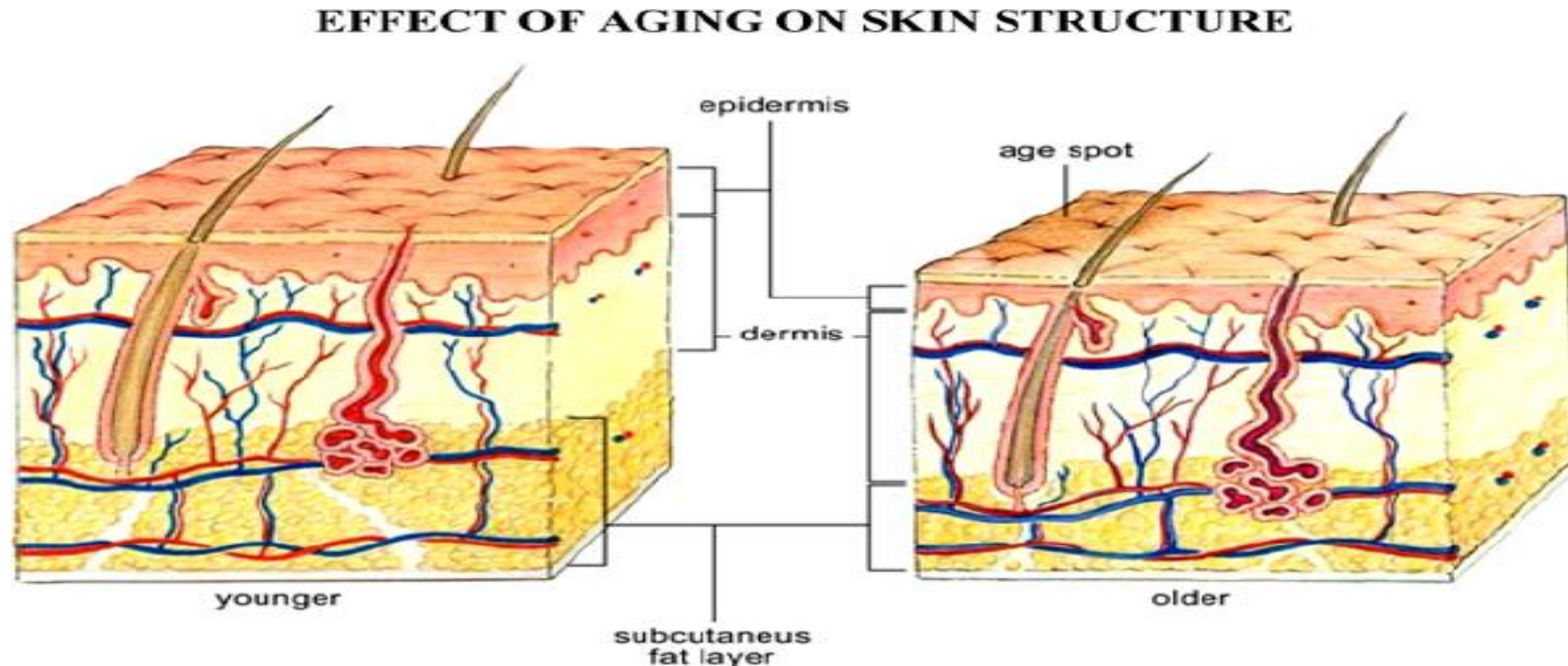

Skin Protocol

TREATMENT PROTOCOL : 3 SESSIONS

Maintenance Protocol – 1x 12/mo

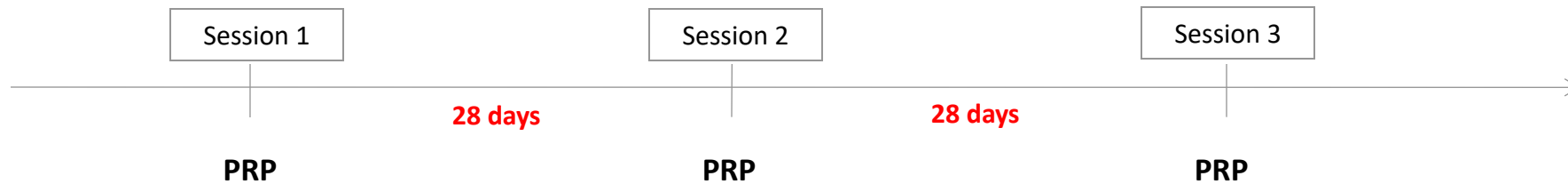

IN ONE PROCEDURE

face, neck and décolleté

disinfect with an aqueous solution

inject **PRP** – papulae and nappage
OR USE ROLLER

BENEFITS

- TIRED ,PHOTO DAMAGED SKIN
- IMPROVE HYDRATATION,
- DARK SPOTS AND CAPILIARS
- RADIANCE
- LIFTING

Face and skin rejuvenation

With stem cell therapy



Lifts forehead creases

Tightens eyebrows

Reduces frown lines

Lightens fine lines

Lifts and firms jowl area

Lightens smile lines and reduces nasolabial folds

Removes dark spots

Improves neck wrinkles

EXCELLENT NATURAL RESULTS



EXCELLENT NATURAL RESULTS

Acne scar



EXCELLENT NATURAL RESULTS

PRP treatment - 1 session



EXCELLENT NATURAL RESULTS

Courtesy of Janine Ball



EXCELLENT NATURAL RESULTS

Courtesy of Janine Ball



EXCELLENT NATURAL RESULTS

Courtesy of Janine Ball



EXCELLENT NATURAL RESULTS

Courtesy of Janine Ball



**EXCELLENT NATURAL RESULTS
HAND REJUVENATION**



SLAPPY ENDING

Baldness cure on the way after scientists use stem cells to grow 'unlimited' hair

seen in National Press, June 2019

MILLIONS of slapheads could be cured of baldness after scientists used stem cells to develop a way of making "unlimited" hair.

Replacement locks could be grown from a person's blood sample. One researcher told a summit: "This is a critical breakthrough."



UK hair loss expert David Fenton said: “This research is very interesting.”

Currently only two drugs, minoxidil and finasteride, are available to treat the condition.





Stem Cells + PRP

➤ Concentration of Stem Cells, examples

- Bone Marrow
- Fatty tissues
- Hair Follicle

Hair anatomy

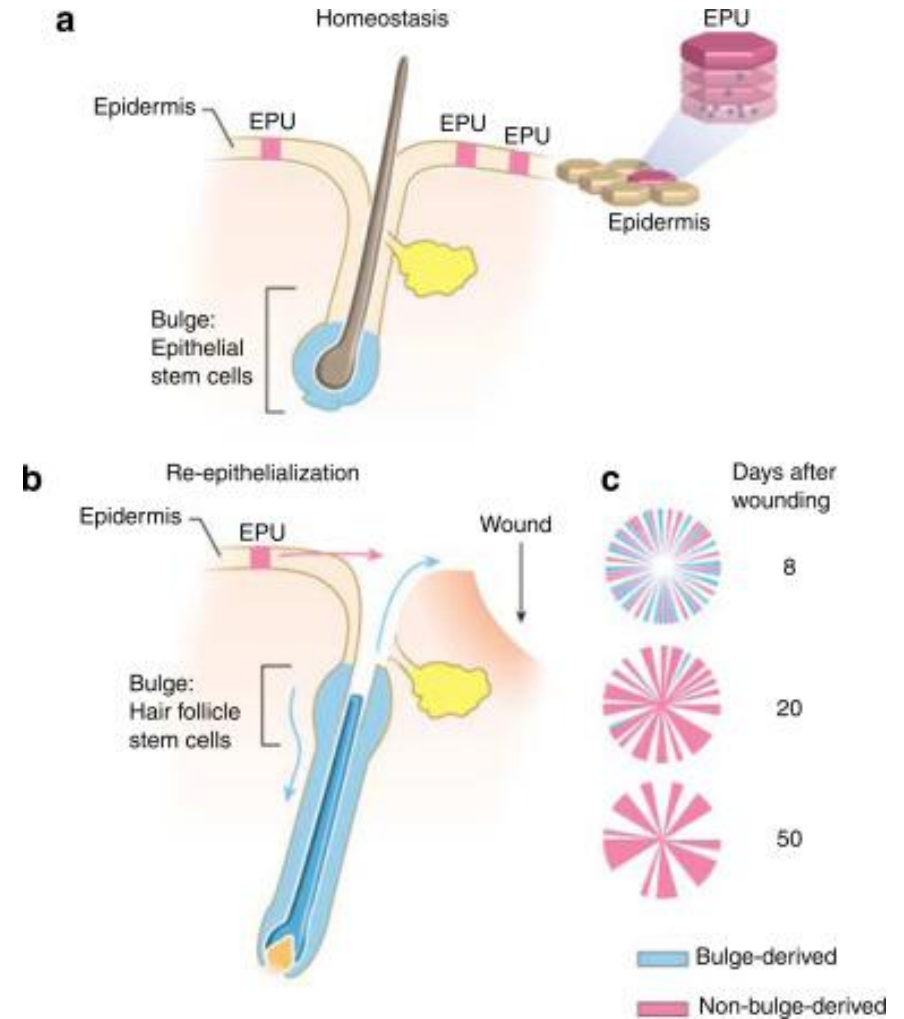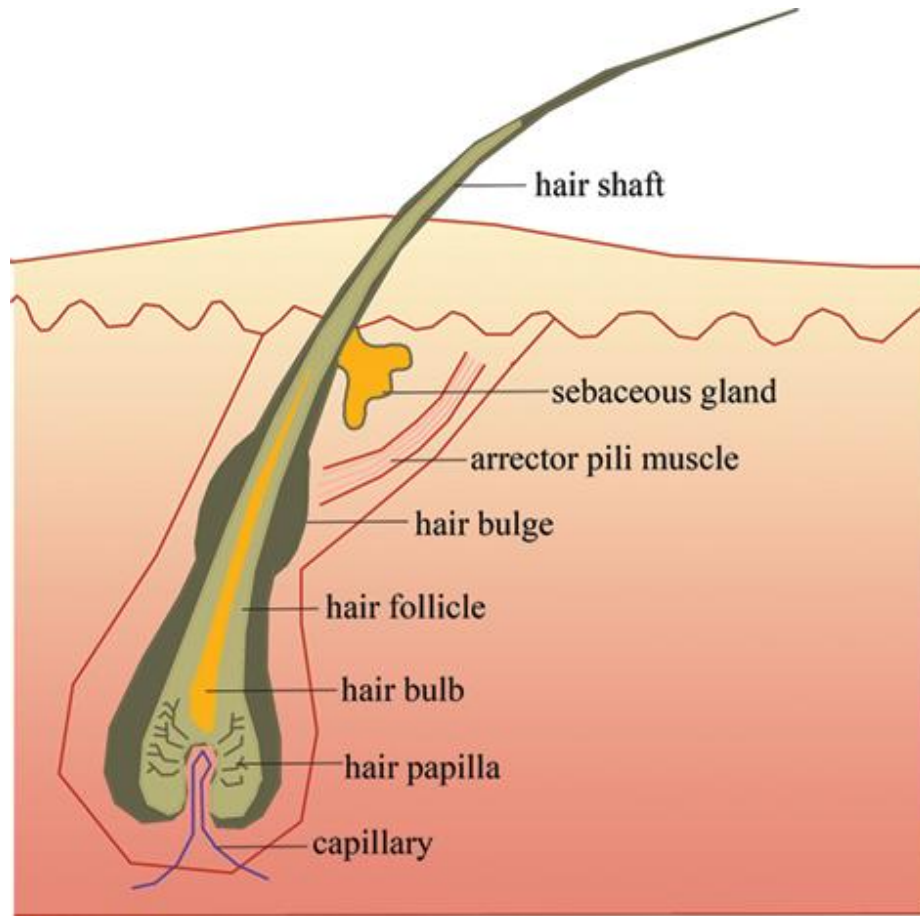

Role of GF

PLATELET



PDGF

**Platelet Derived
Growth Factor**

Cell Growth, new generation and repair of blood vessels, collagen production



VEGF

**Vascular Endothelial
Growth Factor**

Growth and new generation of vascular endothelial cells



FGF

**Fibroblast
Growth Factor**

Tissue repair, cell growth, collagen production, hyaluronic acid production



EGF

**Epithelial
Growth Factor**

Promotion of epithelial cell growth, angiogenesis, promotion of wound healing



TGF-β

**Transforming
Growth Factor**


Growth and neogenesis of epithelial cells and vascular endothelial cells, promotion of wound healing

Hair Restoration Hope

Guardian 07/2022



Hair cycle

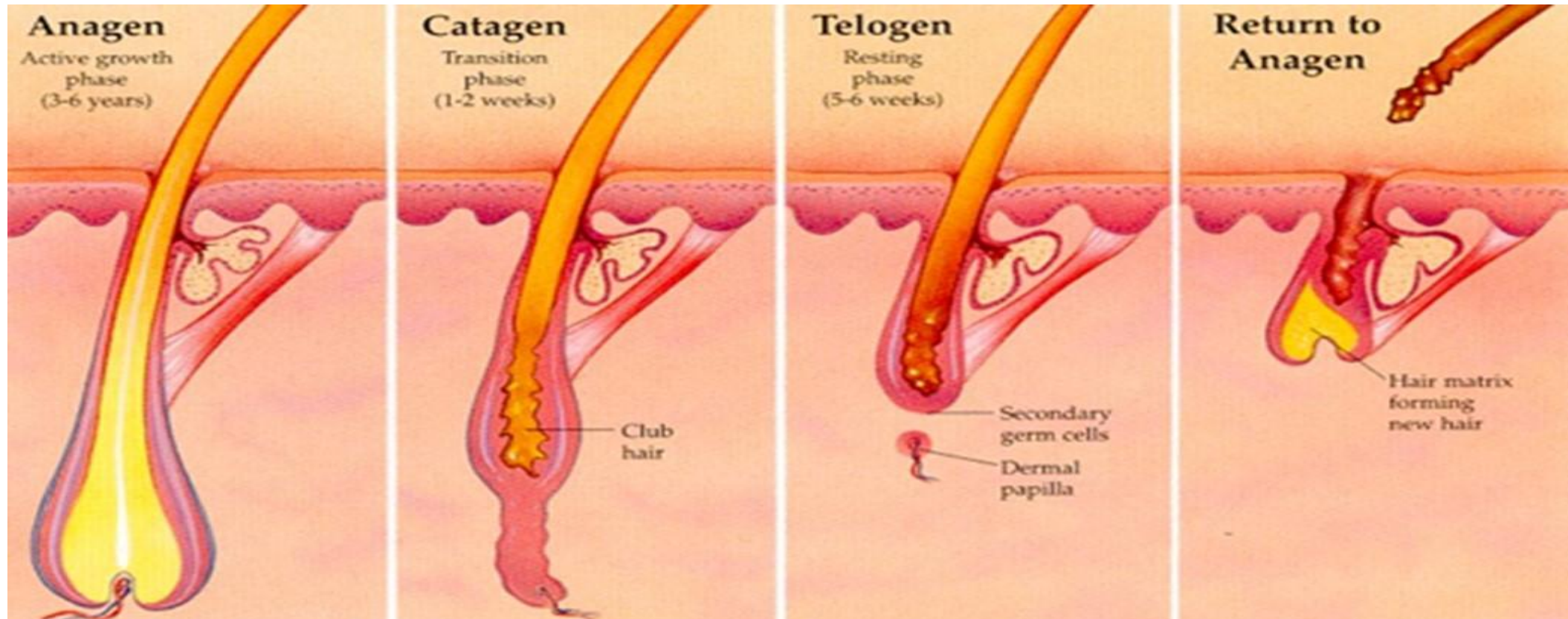

Alopecia types

- **Androgenetic Alopecia (Pattern Hair Loss)**

Male pattern hair loss and female pattern hair loss. Hairs falls out in a well-defined pattern in men, whereas in women, the hair usually becomes thinner all over the head rather than receding from the temples.

- **Frontal Fibrosing Alopecia (Scarring Alopecia)**

Frontal Fibrosing Alopecia (FFA) is a type of Scarring Alopecia, where the hair follicle is destroyed and replaced with scar tissue. In Frontal Fibrosing Alopecia, hair falls out across the front of the scalp. Hair loss can also affect the eyebrows.

- **Chemotherapy Induced Alopecia (Anagen Effluvium)**

Chemotherapy-Induced Alopecia (CIA) is when the hair falls out following chemotherapy. It is one of the most visibly distressing side effects of commonly administered chemotherapeutic drugs.

- **Telogen Effluvium**

With Telogen Effluvium more hairs than usual move into the telogen (resting) phase and shed, so you may notice more hair falling out than usual. Telogen Effluvium is often caused by a physical or psychological trigger and often resolves itself spontaneously.

- **Alopecia Areata**

Alopecia Areata is thought to be an autoimmune condition that causes hair to fall out, usually in usually in round or oval patches on the scalp or other places on the body that grow hair, such as the beard, eyebrows or eyelashes. Types of Alopecia Areata include Patchy Alopecia Areata, Alopecia Totalis, Alopecia Universalis, Alopecia Barbae, Diffuse Alopecia Areata and Alopecia Ophiasis.


- **Traction Alopecia (hair loss)**

Traction Alopecia is when hair falls out because it has been pulled in the same way for a long time. This condition is often caused by tight hairstyles, relaxers or extensions.

Treatment protocol :Alopecia



MESURE THE HAIR GROWTH / EVERY 2-3 MONTHS

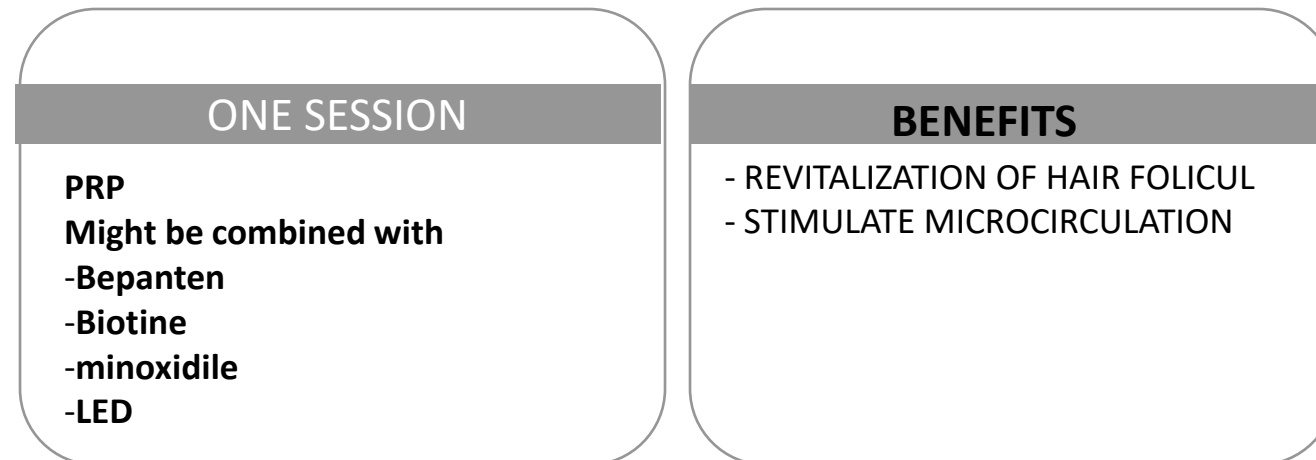

EXCELLENT NATURAL RESULTS

Hair restoration



Hair regrow




Hair regrow

BEFORE PRP TREATMENT



AFTER 4 TREATMENTS

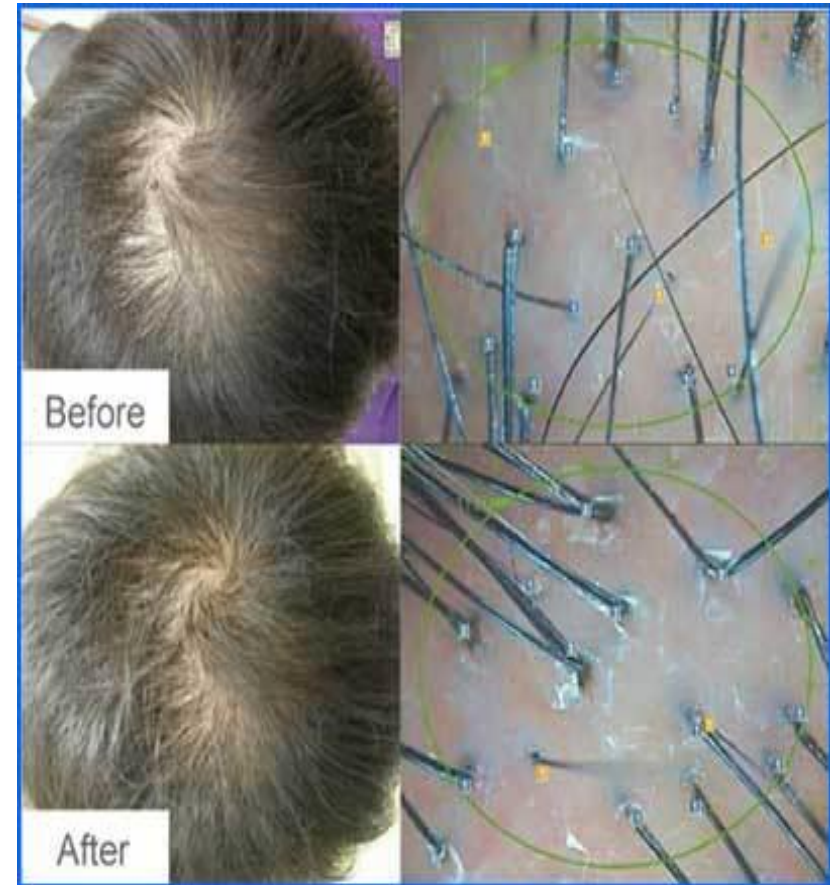

Hair regrow



CONTRA INDICATION FOR PRP THERAPY

- Active inflammation process
- Low platelets number
- Coagulation factors deficiency
- Anti coagulation therapy
- Carcinomas -5 years diseases free
- Autoimmune diseases
- Pregnancy
- Hyper cholesterolemia

PRP Pure has partnered with Pabau

The Clinic Management Software trusted by over 2000 clinics worldwide to run their entire business from one easy-to-use system.

Key Features



Paperless Medical Forms



Payments & Accounting



Online Bookings



Before and After Photos



Automatic Reminders



No Show Protection

As part of our partnership with Pabau, our [PRP consent form](#) and [aftercare information](#) are [digitally available within the system](#), ready to be used in your clinic.

Need to reorder your [PRP Pure Kit](#)? Use the [stock management feature](#) and seamlessly place an order when the stock goes low.

Scan the code and enjoy this offer available exclusively
through our partnership

Enjoy free use of the software **until you reach 50 clients**

Or

Have more clients?

Enjoy a **3-month free trial**



Limited offer: Earn a **£200 voucher** to spend on live 1-to-1 training when you activate your free trial.



Pharma Hermetic



COMPANY INTRODUCTION

Pharma Hermetic is one of the pioneers in hair biotechnology products from Barcelona, Spain.

It's a specialty pharmaceutical company producing hair care and cosmetic products.

Pharma Hermetic was born after an extensive experience of more than 20 years in the cosmetic and pharmaceutical world, Pharma Hermetic founders decided to combine their knowledge and the acquired know-how to establish a company that focuses its activity on the hair care.

Family run business with more than 20 years bioscience and pharmacology background and history.

Presence in over 40 countries.



Company objectives:

Pharma Hermetic manufactures abiding pharmaceutical standards using 100% sterile, organic ingredients.

With competitor products, ingredients like seawater, or growth factors, peptides, come from biotechnological procedures, giving Pharma Hermetic the added value, differentiating us in regard to other companies within the industry.

Our products are manufactured under strict GMP standards, with EU approval, registered with the CPNC.

Pharma Hermetic products are:

- Dermatologically tested
- Clinically tested
- Intolerance tested



AS FEATURED IN

Men's Health

Daily Mail

LUX
life

platinum
live, look & feel fabulous

aesthetic
MEDICINE

THE Sun

Journal of
AESTHETIC
NURSING

COSMOPOLITAN

HAIR RECOVERY PROGRAM SP55



Pharma Hermetic UK
Health Beauty and Wellness Awards 2021
WINNER

HAIR RECOVERY PROGRAM SP55



HAIR RECOVERY PROGRAM SP 55

- The [Hair Recovery Program](#) SP55 is a cosmetic topical solution designed for people suffering with hair loss that works from scalp level to support the stem cells in the follicle.
- Our products increase life length of hair stem cells and also balance scalp nutrition.
- The products also reduce inflammation of the hair follicle and provide a better supply of key ingredients for a healthy hair growth.



Inflammation
to be treated
for effective
results in
AGA

INFLAMMATION

- Restricts follicle growth
- Decrease oxygen and nutrient supply to AGA-prone tissues
- Leads to tissue degradation
- Speeds up hair follicle miniaturization
- Eventually AGA speeds up

Minoxidil, and *5 α reductase inhibitors*



The drawbacks of medical choices:



Every hair loss patient is not the right candidate



Life time dedication



Re-bound effects after stopping the treatment



Side effects not always welcomed



Injectable methods are sometimes painful

HAIR RECOVERY PROGRAM SP55

KEY INGREDIENTS

- Chain of 8 Peptides
- 150 + amino acids
- 100% sterile, organic natural ingredients – Biotin, Caffeine, Vitamins +++
- The mechanism of action of **HRP** SP55 ingredients is to block the DHT + increase the activity of the Telomerase in the stem cells of the hair.

TO:

- Control and decrease inflammation
- Activating anti- oxidant activity
- Remove free radicals
- Relax blood vessels for more oxygen and nutrients
- Shorten the delayed growth of hair follicles



HAIR RECOVERY PROGRAM SP55

How Do Peptides Work for Hair Growth?

Peptides are involved in many physiological processes. Their acceptance as natural molecules, relatively high stability and well-defined actions have made them attractive for many skin and hair related indications, most notably in anti-aging and anti-hair loss therapy.

Peptides of course occur in every cell in the human body, have links to every organ, including those responsible for the hair growth cycle, especially the hair follicles. The follicles need protein to produce hair. Without the presence of adequate protein in the body, it will be difficult to achieve a balanced hair growth cycle.

Peptides, being “smaller proteins,” when applied, cause changes in cell behavior, stimulate follicle growth, and promote natural hair production and sometimes improvement of hair color.



HAIR RECOVERY PROGRAM SP55

The biological activity of HRP key ingredients have been analyzed in depth. **HRP SP55** ingredients prevent telomerase inhibition mediated by different chemotherapeutic agents and decreases oxidative stress protecting cells from DNA damage and delaying the induction of cellular senescence.

These findings support the certainty that the treatment with HRP SP55 ingredients could prolong the lifespan of cells in hair and skin.




HAIR RECOVERY PROGRAM SP55




Increases the
TELOMORASE
activity




Enhances the growth
of new hair

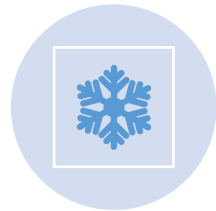


Activates dormant
follicles

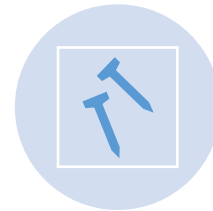

Repairs the
weakened and brittle
hair

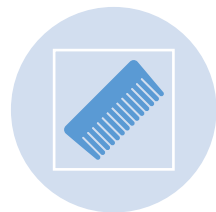

Thickens and
strengthens the
existing hair

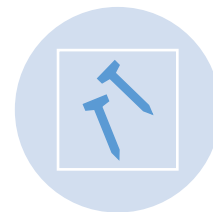

Blocks DHT
formation

HAIR RECOVERY PROFFESIONAL LINE



Ampules (2ml)

- Suitable for home / professional use
- Once every 4 - 10 days
- Topical application / roller
- Home / salon / clinic treatment



Vials (5ml)

- Suitable for clinic use
- Ideally once every 10 days
- Topical application
- At least half vial every 7-10 days - applied with roller



HAIR RECOVERY HOME USE



Hair Mask (250ml)

- For dry and damaged hair
- Home Treatment
- Improves hair quality
- HA + Natural Butters & Oils



Hermetic Lotion (50 ml)

- Daily use
- Complements vials & Amps
- Home treatment



Hermetic Shampoo (200ml)

- Support the treatment
- Natural DHT blockers
- Home Treatment
- Improve hair quality
- Strengthen hair condition





Our experience:

HRP SP55 topical solution helped our patients to

- Reduce scalp inflammation
- Boost better quality hair growth
- Stimulate hair production
- Thicken the existing hair
- No side effect
- No rebound effect

 **PHARMA**
HERMETIC

REAL CASES – Thricoscope 8 weeks treatment



REAL CASES



PHARMA
HERMETIC



PHARMA
HERMETIC

REAL CASES



PHARMA
HERMETIC



PHARMA
HERMETIC

REAL CASES



REAL CASES



PHARMA
HERMETIC



PHARMA
HERMETIC

REAL CASES



REAL CASES



Product Prices



- HRP Vials 5 X 5ml
- £75 - cost
- 5-10 Treatments
- £80-120 per treatment



- Derma Roller
- £5 – cost
- Single use



- Pharma Hermetic Shampoo
- £26 - cost
- RRP £44



- Scalp Analyser
- £150 - cost



- HRP Home Lotion
- £75 - cost
- RRP £145



- HRP Ampules 10 X 2ml
- £70 - cost
- RRP £199



- Mask
- £22 - cost
- RRP £42

Support



PHARMA HERMETIC

HAIR RECOVERY PROGRAM

By Pharma Hermetic
Cutting edge technology

Hair Restoration RESULTS IN 8 WEEKS

- 100% Natural Ingredients
- For All Hair Types
- DHT Blocking
- No Side Effects
- Painless Topical Application
- Provides Nutrients
- Lasting Results

Clinic Treatments & At-Home Treatment Available from your clinic

Treatment For Men And Women



POSTER

Shampoo

- Natural ingredients
- Gentle to hair
- Trace Elements and Vitamin C
- All hair types
- Professional strength

How to Apply:

- Apply to wet hair
- Massage and leave it on your hair for 5 minutes
- Rinse with plenty of warm water

Dermatologically Tested



Lotion

- Professional strength
- Advanced restoration hair system
- Stimulates the cells

How to Use:

- Shake before using
- Apply on dry hair
- Do not wash your hair for 8 hour after applying the product
- Avoid sunlight for 8 hours

Initial Treatment – daily application for 60 days

Maintenance – one application per week



PHARMA HERMETIC

THE JOY OF HEALTHY HAIR

CONTACT

📱 @pharmahermeticuk
✉ info@pharmahermetic.co.uk
🌐 www.pharmahermetic.co.uk

CLIENT LEAFLET

With application protocol

- All stock images
- Description
- Before / After
- SEO words
- ETC.

PHARMA HERMETIC

Contact details for training and further info

- Contact Pharma Hermetic
- info@pharmahermetic.co.uk
- 0208 680 2323

- Contact PRP PURE
- info@prppure.com
- 0208 680 2323